OccuniX

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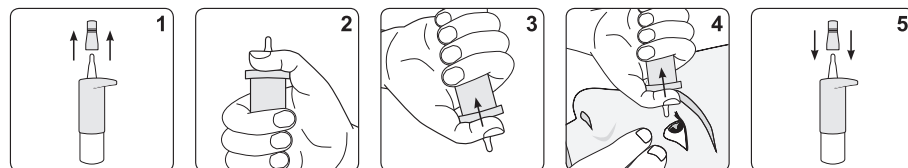
OccuniX

GEL EYE DROPS

With 3 mg/ml Sodium hyaluronate and Dexpanthenol. Preservative-free.

Instructions for use

Please read this information carefully because it will help you using this product properly.

Composition

Sodium hyaluronate 3 mg/ml, dexpanthenol, sodium chloride, potassium chloride, disodium phosphate dodecahydrate, sodium dihydrogen phosphate dihydrate, magnesium chloride hexahydrate, calcium chloride dihydrate, sodium hydrogen carbonate, purified water.

Indication and clinical benefit

For intensive lubrication of the eye surface in case of dry, irritated, strained or burning eyes resulting from dry eye syndrome and environmental factors (e.g., medications, pollution, screen use, air conditioning, allergy, wetting contact lenses during wear) as well as after eye surgery. OccuniX makes wearing contact lenses more comfortable.

Performance characteristics

Preservative-free aqueous solution based on hyaluronic acid and electrolytes. Due to its high water-binding capacity, the eye is intensively moisturised and additionally cared for by the combination of hyaluronic acid and dexpanthenol.

Application

1. Remove the protective cap before (Fig. 1) and replace the cap immediately after each use (Fig. 5). Avoid any contact with the dropper tip. Before the first application press down the dropper until the first drop appears. Now the bottle is ready for use.
2. Place your thumb on the ergonomically formed thumb rest and wrap your other fingers around the lower part of the bottle (Fig. 2).
3. Tilt your head back a little and turn the bottle vertically upside down above your eye (Fig. 3).
4. Carefully open your eye using the thumb and index finger of your free hand or gently pull the lower lid down (Fig. 4).
5. Instil 1-2 drops onto the surface of the eye or into the conjunctival sac by pressing the thumb rest (Fig. 4).

We recommend using these drops 4 times per day. As they are preservative-free, the eye drops can be used more frequently without hesitation. Overdose is not possible due to the method of application and composition. OccuniX can also be used while wearing hard or soft contact lenses.

Contraindications and Precautions

Do not use in case of hypersensitivity to any of the ingredients.

If additional eye drops/ eye ointments are used, a time interval of about 30 minutes should be kept between the applications.

If discomfort persist or getting worse, consult a healthcare professional.

Keep out of reach of children.

Side-effects

In very rare cases hypersensitivity reactions may occur, which subside quickly and without consequences after discontinuation.

In case of any serious incident that has occurred in relation to this medical device, report to the manufacturer or your competent authority.

Notes

Your vision may be temporarily affected immediately following the use of the product. For adults and children.

Children must be supervised by an adult.

In very rare cases, calcification of the cornea in association with the use of eye drops with a high phosphate concentration has been reported in patients with significant damage to the cornea. OccuniX contain only a physiological concentration of phosphate so that their use is in principle not restricted even in case of existing severe corneal disorders / damage. If other eye drop preparations are also used, however, their phosphate concentration should be borne in mind in this regard since, particularly in the case of anti-glaucomatous agents or anti-allergy agents, they may contain high phosphate concentrations.

Shelf-life, storage and disposal

Recommended storage: room temperature, do not freeze.

Can be used up to 6 months after first opening.

Do not use after expiry date.

Disposal according to local requirements.

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