Parasine Rapid

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EN Parasine Rapid:

Please read the instructions for use carefully as they contain important information.

Parasine Rapid is a medical device. Follow the instructions for use carefully. Keep the instructions for use in case you need to read them again. If you have any questions, or if you need further information, please do not hesitate to contact your pharmacist.

Intended purpose: The medical device Parasine Rapid is indicated for human use to eliminate lice in the patient's hair.

Ingredients: Isopropyl myristate, Cyclopentasiloxane, Dimethicone, Nerolidol, Melaleuca Alternifolia Leaf Oil, Thymus Vulgaris Oil.

Characteristics: The main active substance in medical device Parasine Rapid is isopropyl myristate and cyclomethicone D5. Isopropyl myristate ensures the physicochemical disruption of the waxy layers of the exoskeleton of lice with their subsequent dehydration and death. Cyclomethicone D5 prevents air from getting to the lice and their eggs, leading to their subsequent suffocation and/or rupture. Tea tree and thyme essential oil provide an additional function by preventing the scalp irritation that accompanies the occurrence of head lice.

Indications: Parasine Rapid is indicated to eliminate lice and their eggs in the patient's hair.

Contraindications: Hypersensitivity to any one of the substances stated in the ingredients of the product.

Interactions: No undesirable interactions are known for any of the product's ingredients. If other products are used, this topical product may be washed out when using Parasine Rapid.

Adverse effects: The product is usually very well tolerated. Skin irritation may occur in rare cases, manifested through itching or redness. If any of these adverse effects occur to a serious extent, or if you notice any adverse effects that are not mentioned in these Instructions for Use, please contact a doctor or pharmacist. Any serious adverse effect that occurred in connection with the use of this medical device should be reported to the manufacturer and to the competent national authority.

Dosage and method of application:

Parasine Rapid can be used for children from 2 years of age and adults, including pregnant and nursing women. Children under 15 years of age should use the product under adult supervision. Do not use the product in the event of hypersensitivity to any of the product's ingredients. The product is intended for external use only.

- Before applying the product, check for the presence of lice in the hair of all family members. It is vital to treat everyone with a proven infestation of lice at the same time in order to prevent re-infestation.
- Cover the person's shoulders with a towel.
- Unscrew the applicator cap. Apply the product to dry hair by repeatedly squeezing the bottle, moving from the forehead to the top of the head, then continue to the ears and nape along the entire length of the hair. The amount of solution will depend on the thickness and length of the hair and the degree of infestation. For shoulder-length hair, you should use approximately 50 ml of the product.
- After application, rub the solution thoroughly and evenly into the hair so that it is well moistened along its entire length, including the scalp.
- Leave on for 10 minutes after applying the solution.
- After the minimum requisite time has elapsed, comb the hair with a fine-toothed comb along individual strands.
- Remove any parasites from the comb as you go.
- After combing, rinse your hair with lukewarm water and then wash with shampoo.

Regularly check the hair for the occurrence of lice and/or nits over the next 10 days. Immediately repeat the procedure if lice/nits are present or if you are unsure whether they are alive or dead.

Repeat the procedure after ten days even if no live lice and/or nits were found when examining the hair. This will prevent re-infestation in the event an examination failed to find any lice or nits that might have survived.

As an auxiliary measure that effectively reduces the risk of re-infestation, we recommend that you also check all members of the family at the same time. It is further recommended to wash clothes, bed linen, towels, etc. at a temperature of 60 °C. Combs and similar items can be decontaminated by placing them in water at a temperature of 60 °C for 10 minutes. Unwashable items that need to be decontaminated should be placed in a plastic bag and left in the refrigerator for 5 days, or at room temperature for 3 weeks. Clean your place of residence thoroughly, in particular the carpets, upholstery, etc.

Caution:

- ⚠ Upon accidental ingestion, drink plenty of water immediately and if you have any further problems, please contact a doctor.
- ⚠ If the solution gets into your eyes, rinse with water immediately for about 10 minutes.
- ⚠ If there is any skin irritation (redness, pain, itching) contact a doctor immediately.

Storage: Keep out of the reach and sight of children. Keep in the original packaging.

Warning:

- ⚠ Do not apply the product to broken skin.
- ⚠ If it enters a wound, wash with water or a readily available disinfectant.
- ⚠ Do not use the product after the expiration date.

Packaging: Parasine Rapid is supplied in a PE bottle with a cap. The bottle contains 100 ml or 200 ml of solution. Both packagings may not be available on the market.

Disposal: Return any unused product to the pharmacy. Dispose of the clean packaging material in the appropriate recycling bin.

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