CS Otic Solution:

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8 5 9 4 0 7 1 2 7 0 0 9 4

EN Otic Solution:

**Carefully read the Instructions for Use before using the device. Keep the Instructions for Use. You may need to read it again.**

**Intended Purpose:** The medical device Otic Solution is intended for human use by patients suffering from problems with the external ear canal, for the prevention of illnesses affecting the external ear canal and for the treatment of hardened sebaceous plugs (cerumen) by mechanically rinsing the ear, softening, and dissolving earwax while hydrating, soothing, and removing water, dust and impurities from the ear canal through physical/chemical action. Otic Solution has positive effects while regenerating of the skin of the external ear canal.

**Ingredients:** Glycerin, Butylene Glycol, Choline Salicylate, Alcohol, Lavandula Angustifolia Oil, Herbarum extractum ex Calendulae flos cum calyce, Verbasci flos et Hyperici herba, Thymolum.

**Content of Medicinal Agents in 1 g of the Device:** Choline salicylate 80%: 174 mg, Herbal extracts: 82 mg (prepared from Calendulae flos: 5.25 mg, Hyperici herba: 3.53 mg and Verbasci flos: 3.53 mg), Thymol: 1 mg. This medical device does not contain derivatives of human blood.

**Characteristics:** Otic Solution is an otologic product - ear drops in solution form. It is a slightly yellowish oily liquid contained in a bottle with an applicator.

The device contains a mixture of **glycerin and butylene glycol**, which has a soothing effect and softens and dissolves hardened earwax. It is also serves as a mixture of auxiliary agents in which other elements are dissolved. Together, these create an environment unsuitable for the growth of bacterial infection.

This product contains the following medicinal agents:

**Cholin salicylate 80%** is used in the product in the amount of 17.4 % (w/w). It has anti-inflammatory effects that supplement the effects of the device when hardened cerumen, contaminants, or the mechanical removal of impurities causes irritation to the skin of the ear canal.

**Herbal extract** ex Calendulae flos cum calyce, Verbasci flos et Hyperici herba is an ethanol extract mixture of herbs (Calendula flower, mullein flower, and St. John's Wort). It is used in the amount of 8.2 % (w/w). It aids in regeneration and stimulates the self-cleansing properties of the skin.

**Thymol** is a device used in the amount of 0.1 % (w/w). It is a pure substance, 2-isopropyl-5-methylphenol. Thymol has anti-bacterial effects. It favorably supplements the effects of the device, especially when the ear canal retains water, which could create an environment suitable for the growth of microbial infection.

**Levander oil** has anti-bacterial effects and also adds a pleasant scent to the device.

**Indications:** This product is used to remove impurities from the external ear drum and prevent the creation or development of bacterial infection and inflammation of the ear canal, provided that the ear drum is undamaged.

**Contraindications:** Hypersensitivity to any of the product's listed ingredients, damage to the ear drum in the form of an infection or traumatic effect (perforated ear drum).

**Interaction:** The interaction of the product is unknown.

**Undesired Effects:** Risk of local reaction in the form of an allergy, irritation, or hyperaemia of the ear canal exists in rare cases.

**Dosage and Application:** An excessive amount of earwax can build up when contaminants (water, dust) enter the outer ear canal or its self-cleansing properties are impaired. This could lead to a worsening of the ear canal's passability and the formation of conditions for the multiplication or growth of microorganisms that can cause infection and other health problems.

Otic Solution is produced in three variants, namely:

**EAR DROPS:** for application using the pipette included in the cap, apply 2-3 drops to the opening of the ear canal.

**EAR SPRAY:** for application using the mechanical sprayer (spray), first press the sprayer several times to draw the liquid in. Then insert the end of the sprayer into the opening of the ear canal and press 2-3 times.

**EAR DROPS/SPRAY:** for application using the pipette included in the cap, apply 2-3 drops to the opening of the ear canal. If you prefer using the mechanical sprayer (spray), remove it from the protective packaging and screw it firmly to the bottle instead of the cap with the pipette. First press the sprayer several times to draw the liquid in. Then insert the end of the sprayer into the opening of the ear canal and press 2-3 times. Do not use the removed pipette.

Usually apply 2-4 times per day into every ear. Prior to use, heat the solution to body temperature (36-37 °C) by warming in the hands, for example. Proceed with cleaning the ear no sooner than 10 minutes after using the device. For mechanical cleaning of the ear, carefully use a Q-tip as the sebaceous plug can be difficult to access and there is a risk of pushing it deeper into the ear canal. Continuous use should be limited to 5 days.

**Caution:**

- ⚠ Do not use the device if there is damage to the ear drum.
- ⚠ If accidentally swallowed, drink a large quantity of water and contact your physician.
- ⚠ In case of direct contact with eyes, flush eyes under running water.

**Storage:** Keep out of the reach of children. Keep in the original packaging. Close tightly after use and store in a vertical position.

**Warning:**

- ⚠ Do not use this device if you are allergic or hypersensitive to any of its ingredients.
- ⚠ If you observe any undesirable side effects, consult your physician.
- ⚠ If you choose the mechanical sprayer for application, discard the removed pipette.
- ⚠ Pregnant or nursing women and children younger than 3 years should only use the device following consultation with a professional physician.
- ⚠ Children younger than 15 years should only use the device under adult supervision.

**Packaging:** An Otic Solution package contains 10 ml of solution in a brown glass bottle and includes an ear applicator (a pipette, spray or a combination of the two). Not all variants may be available on the market.

**Disposal:** Return any unused product back to the pharmacy. Dispose of clean packaging into a suitable recycling container.

ENEO Pharmaceuticals s. r. o., Pražská 442, 281 67 Stříbrná Skalice, Czech Republic, www.eneo.cz, e-mail: info@eneo.cz

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